5968

5969

H

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GB

OPERATING MANUAL

Digital column scale – Model 5969

With height rod – Model 5968

UA

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Digital column scale – Model 5969

With height rod – Model 5968

Thank you for purchasing a Momert product. We hope you will be satisfied with our product throughout its service life.

Please study the entire Operating Manual carefully before you start using the product. Keep the manual in a safe place for future reference. Make sure other people using the product are familiar with these instructions.

Technical parameters

Voltage	4 x 1.5V AAA batteries or 5V adapter (included)
Max. measurable weight	200kg
Division	100g
Min. measurable weight	2kg
Weighing units	kg, lb, st
Display size	LCD diagonal = 8,0cm; 3,15 in
Max. measurable height	210 mm / 82,5 inch
Division	0,5 cm / 0,25 inch
Min. measurable height	81 mm / 32 inch
Units	cm, inch
Working/Storage temperature	10-40°C/-10-50°C
The scale can operate in the temperature range between +10-40°C, but provides the most accurate measurement results at +22°C.	

Model 5968

PARTS OF THE PACKAGE:

- 1pc scale with black mat
- 1pc display holder column with display *and at Model 5968 with height rod*
- 2pcs display holder column fixing screws
- 1pc hex key
- 4pcs 1.5V AAA batteries: **Batteries are not included in warranty.**
- 1pc 5V mains adapter with USB (A type - B type) cable: **USB cable is not included in warranty.**
- 1pc operating manual

- 1pc warranty card

SPECIFICATION:

- BMI function
- Height rod – *at Model 5968*
- Precision measuring system
- Extremely high measuring capacity
- Tare function
- Overload indicator (oL-E)
- Low battery indicator (Lo)


IMPORTANT SAFETY INSTRUCTIONS:

- Make sure you follow the marked polarity of battery when inserting and replacing it.
- If the appliance will be out of use for a longer time, remove the battery.
- Do not leave the unit unattended when in use.
- This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety. Children should be supervised to ensure that they do not play with the appliance.
- Use the product only for the intended purpose specified in the Operating Manual.
- Use the unit only on a stable horizontal surface.
- Before cleaning, assembly, or disassembly of the cover, switch the unit off and remove the battery!
- Do not use abrasive or chemically aggressive substances for cleaning the unit.
- Do not use the unit if it does not operate properly, or if it has been dropped, damaged or exposed to any liquids. Take the unit to an authorized service center for testing and repair.

- The unit is suitable for domestic use only. It is not designed for commercial operation.
- Keep the unit away from heat sources, such as radiators, ovens etc. Do not expose the unit to direct sunlight or moisture.
- Do not touch the unit with wet or damp hands.
- The product is a fragile measuring device. Prevent it from crashing or falling.
- Do not repair the appliance yourself. Contact an authorized service center.


Failure to follow the manufacturer's instructions may lead to refusal of warranty/repair!

DISPLAY: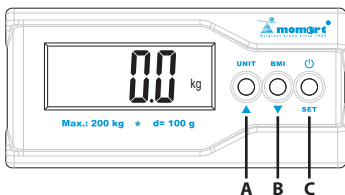

- A** Unit/Value +
- B** BMI function/Value -
- C** On-Off switch/Tare/Height setting

CONNECTORS AND POWER SUPPLY: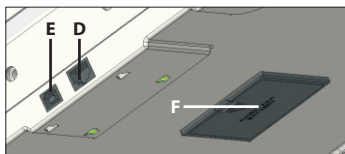

- D** Connecting of the display (UTP connector)
- E** Connecting of the adapter or a PC (USB A connector)
- F** Battery holder for 4 × 1.5V AAA batteries

THE OPERATION OF THE SCALE:

1. Switch on the scale by pressing the  (C) button.
2. If the display shows "0.0", you can start the measuring process
3. If the load level (approx.) is unchanged for 3 seconds, the measured value flashes twice and the display shows the actual weight which will be unchanged even if the load changes.
4. If the load is removed of the scale display shows "0.0" again, the scale is ready again for measuring process.
5. Press the  (C) button to turn off the balance.

ADDITIONAL FEATURES:

BMI function:

- The BMI function can be reached right after the measuring process. If the load level (approx.) is unchanged for 3 seconds, the measured value flashes twice and the display shows the actual weight which will be unchanged even if the load changes. Then by pressing the BMI (B) button can the function be activated. Set your height with the help of the ▲ (A) and ▼ (B) buttons, then press the SET (C) button. In this case the scale will show your BMI value and weight alternately.

If the load is removed of the scale display shows "0.0" again, the scale is ready again for measuring process.

Power supply:

- The adapter connector on the scale display box allows the scale to be operated from mains power also. By using an adapter, the scale can measure continuously without switching off, in this case the measured value is continuously displayed.
- The balance can also be operated with a battery. In this case, the scale switches off after 3 minutes following the completion of

the last measurement.

ATTENTION! If you use the scale with an adapter, remove the batteries as they might run out.

THE BATTERIES ARE NOT COVERED BY THE WARRANTY.


Tare:

- By pressing the $\odot$ (C) button shortly during loading the scale is set to 0 and the displayed weight (tare weight) is memorized, and the display shows "0.0" again. You can continue to measuring process, the scale automatically deduct the tare weight from the displayed value.

ATTENTION! The measuring range will be decreased with the tare weight! If you remove the tare weight, its value will appear on the display with a negative sign, in this case by pressing the $\odot$ (C) button shortly you can clear its value, and then the display will show "0.0" again.

Unit change:

- By pressing the UNIT (A) button, you can switch between kg, lb and st at any time.

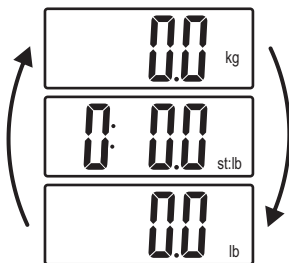ATTENTION!

- Protect the plug input of the mains adapter from damage!
- Do not push the balance sideways under loading process.

CLEANING AND MAINTENANCE:

The surface of the unit may only be cleaned using a wet cloth. Do not use any detergents or hard objects as they may damage the unit's surface!

NEVER clean the unit under running water, do not rinse it, and do not submerge it in water!

REPAIR AND MAINTENANCE:

Any extensive maintenance or repair that requires access to the inner parts of the product must be performed by an expert service centre.

ENVIRONMENTAL PROTECTION:

- Dispose of packaging materials and old household appliances at the collection points for recycling.
- Dispose of the machine's cardboard box in the waste paper container.
- Put the plastic bags (PE) in the plastic waste container.

Appliance recycling at the end of its service life:

A symbol on the product or its packaging indicates that this product should not go into household waste. It must be taken to the collection point of an electric and equipment recycling facility. By making sure this product is disposed of properly, you will help prevent the negative effects on the environment and human health that would otherwise result from inappropriate disposal of this product. You can learn more about recycling this product from your local authorities, a household waste disposal service or in the shop where you bought this product.

ERRORS AND THEIR SOLUTIONS:

Error	Cause	Solution
The scale doesn't turn to zero after turning it on	The scale has been turned on on an uneven surface. After changing the batteries. It was under weight when turning on. It picked up the reference weight incorrectly.	Press the  (C) button again then the scale will be set to 0 and the display will show "0.0".
	If you have tared a weight by pressing the  (C) button, the weight of the tared item will be displayed with a negativ sign, when you remove it from the scale.	Press the  (C) button then the scale will be set to 0 and the display will show "0.0".
	The scale is overloaded and the weight placed on the scale exceeds the maximum value specified in the technical parameters. ATTENTION! The maximum value contains also the tare weight!	Weight should be reduced. ATTENTION! Overloading can cause sensor error!
	The "Lo" signal appearing on the display indicates that the batteries have been exhausted.	Take off the cover of battery holder (F). Replace the old batteries with new ones then click the cover back in place. Dispose of the old batteries to the right place.

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